


# DRUGCHECK®+

## BIOCHEMISTRY

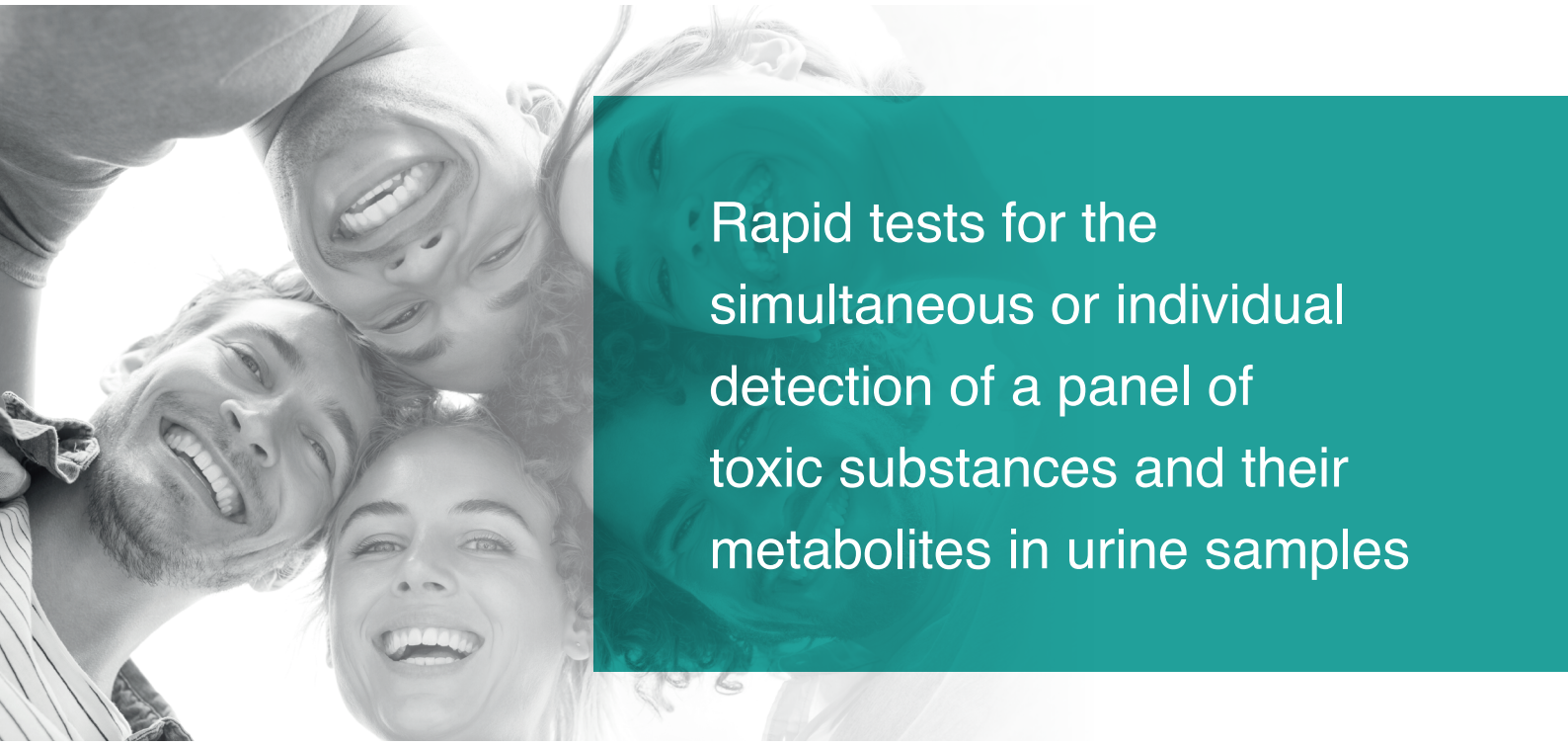

Rapid tests for the simultaneous or individual detection of a panel of toxic substances and their metabolites in urine samples

- *Easy to use & Quick results*  
3 drops of urine are enough for a result in 5 minutes
- *Detection of 18 different parameters in urine*  
THC, COC, MOP, BZO, BUP, MTD, EDDP, FYL,...
- *Positive and negative controls available*
- *Simple reading and interpretation*  
Visual reading or automated reading with the BSX READER®

# DRUGCHECK®+ RANGE

## TEST PROCEDURE

### Easy to use & Quick results

- 1 | Collect urine in a clean, dry container.
- 2 | Using the pipette included in the foil pouch, dispense 3 drops of urine (~120 µL) into each well of the cassette.
- 3 | Interpret the result at 5 minutes.



### DRUGCHECK®+

- Mono or multi parameters test cassettes
- Pipettes included


## TEST RESULTS

### POSITIVE



C control band  
BUT no T test band

### NEGATIVE

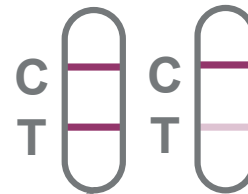

C control band  
AND T test band  
(even weak intensity)

Product name	Parameter(s)	Cut-off (ng/mL)	Reference
DRUGCHECK®+ AMP	Amphetamine	1 000	1140002_SW
DRUGCHECK®+ BUP	Buprenorphine	10	1140004_SW
DRUGCHECK®+ COC	Cocaine	300	1140006_SW
DRUGCHECK®+ COT	Cotinine	100	1140001_SW
DRUGCHECK®+ EDDP	EDDP	100	1140021_SW
DRUGCHECK®+ FYL	Fentanyl	10	1140022_SW
DRUGCHECK®+ MET	Methamphetamine	1 000	1140008_SW
DRUGCHECK®+ MOP	Morphine	300	1140009_SW
DRUGCHECK®+ OXY	Oxycodone	100	1140023_SW
DRUGCHECK®+ PCP	Phencyclidine	25	1140011_SW
DRUGCHECK®+ PPX	Propoxyphene	300	1140024_SW
DRUGCHECK®+ THC	Cannabis metabolites	50	1140013_SW
DRUGCHECK®+ TML	Tramadol	100	1140025_SW
DRUGCHECK®+ 5	AMP, COC, MDMA, MOP, THC		1140014_SW
DRUGCHECK®+ 10	AMP, BAR, BZO, COC, MDMA, MET, MOP, MTD, TCA, THC		1140015_SW
DRUGCHECK®+ 11	AMP, BAR, BUP, BZO, COC, MDMA, MET, MOP, MTD, TCA, THC		1140016_SW
Positive (16 parameters) & Negative controls	Controls for: AMP, BAR, BUP, BZO, COC, EDDP, FYL, MDMA, MET, MOP, MTD, OXY, PCP, PPX, TCA, THC		702003
COT Positive & Negative controls	Controls for: COT		702018

# RAPID TEST READER

## BSX READER®



The BSX READER® is a highly sensitive and robust measurement system for the standardized and automated reading of lateral flow tests of the BSX Range.

The instrument will guide you through the assay procedure by means of a series of messages and animations shown on the display screen.

### *User friendly*

The touch screen guides the user through the different steps of running a test.

### *Full traceability*

Full traceability of user, lot numbers, patients and results to meet laboratory requirements.

### *Standardized reading of results*

Eliminates user error in the interpretation of results.

### *Wide menu of tests available*

Virology:

BIOSYNEX COVID-19 Ag BSX, INFLUENZATOP® BSX, RSVTOP® BSX

Women's health:

AMNIOQUICK® Card BSX, AMNIOQUICK® Duo+ BSX, PREMAQUICK® BSX

Bacteriology:

TETANOTOP® BSX, easyTETANOTOP® BSX

Toxicology:

DRUGCHECK®+ 11/10/5 BSX, DRUGCHECK®+ THC BSX

Product name	Description	Reference
BSX READER®	Instrument + Power cable + Barcode reader + User manual	5060029
DYMO LABELWRITER 450	Printer for use with BSX READER®	502250
LABELS 54*101 MM	Printer labels for BSX tests - 1 roll = 220 labels	131258
LABELS 59*190 MM	Printer labels for DRUGCHECK+ 11/10 BSX - 1 roll = 170 labels	131259



**BIOSYNEX SA**

22 Boulevard Sébastien Brant  
67400 ILLKIRCH-GRAFFENSTADEN  
France

Phone: +33 3 88 78 80 88

Fax: +33 3 88 78 76 78

export@biosynex.com

www.biosynex.com

Please carefully read the instructions  
on the package leaflet and/or on outer  
packaging and the user guide.