# Professional Procedure Guide

**AICNow+ Professional**  
Multi-Test HbA1c System

Note: All instances of the box or pouches in this insert are representative ONLY. Please refer to the packaging included with your specific kit.


### Introduction

- If the temperature label, placed on the outside of every kit, is exposed to a temperature in excess of 50°C (122°F), the dot on the label will turn red and the product should not be used.
- Run the test with all parts of the test kit at the same temperature within the specified range.
- If the kit has recently been at high temperatures (above 28°C/82°F) in the refrigerator, keep the kit at room temperature for at least one hour before use.
- Avoid running the test in direct sunlight, on hot or cold surfaces, or near sources of heat or cold.
- Quality control materials should be used to confirm the test kit is working properly. Refer to the product insert for information on when to run controls.

28°C (82°F)  
18°C (64°F)  
Complete test within 15 minutes.


### Match Lot Numbers

Use analyzer only with the materials included in the original kit. The analyzer will expire after the programmed number of tests have been run. If another test cartridge is inserted, the analyzer will display "00 TL".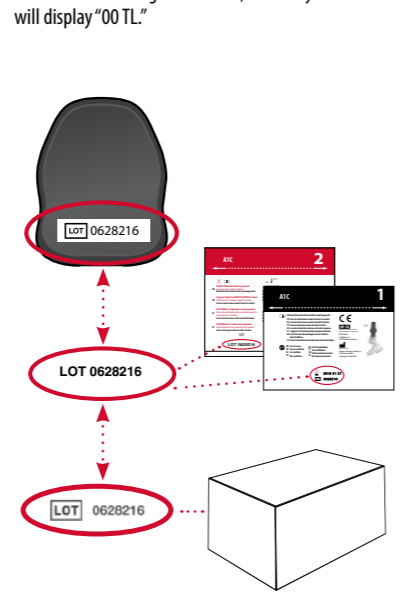

### 1 Open Plastic Shaker Pouch\*

Tear plastic pouch open at the perforation line.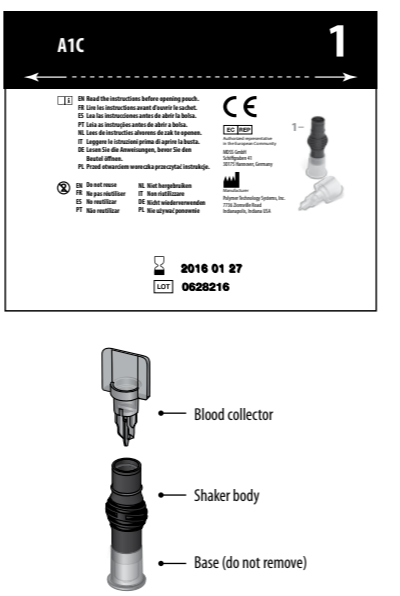

### 2 Collect Blood


**Fingertip method**

- Use your own lancet device to draw blood.
- Gently touch blood drop to fill.


**OR**

**Venous draw method**

- Mix blood well before testing.
- Collect blood from a slide.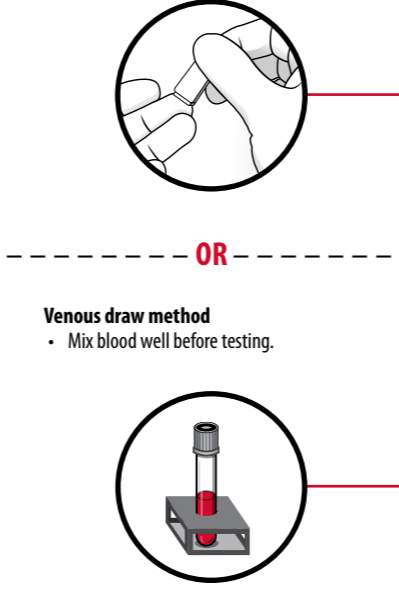


### 3 Verify Blood Amount

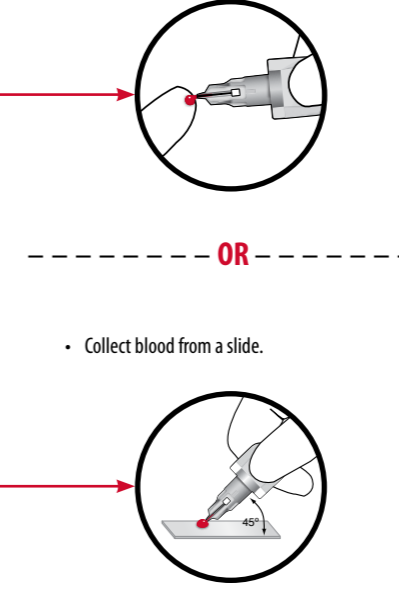No little Add more blood  
Just right  
Too much Wipe away excess

### 4 Insert Blood Collector


Fully insert blood collector into shaker body. Twisting motion helps.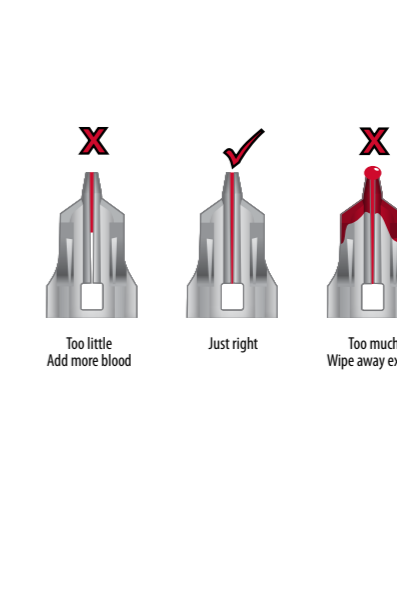

Not fully inserted  
Fully inserted

### 11 Dispose of Cartridge / Save Analyzer

- Dispose of in accordance with your biohazard procedures.
- Save analyzer for additional tests. Analyzer will display, for example, "01 TL" if there is one test left and "00 TL" if you used your last test.
- Analyzer will show result for 15 minutes and will turn off automatically.

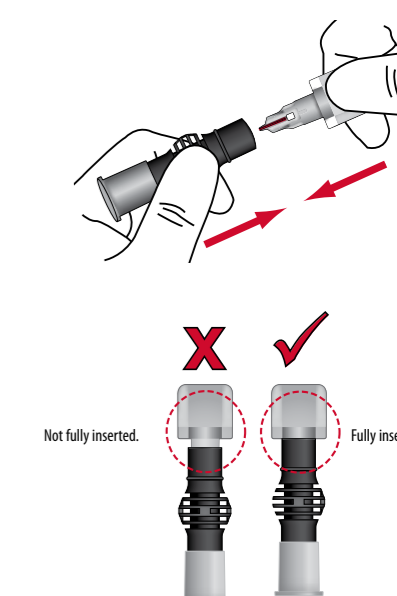NOTE: To run another test, use a new blood collector, shaker, and test cartridge from the same kit and return to Step 1.

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