



A **rapid test** for the qualitative detection of **SARS-CoV-2 antigen** in nasopharyngeal or nasal swabs

- **Easy to use**
 - Using a nasopharyngeal or nasal specimen swab in only few steps
 - Single-dose vials already prefilled with extraction buffer
- **Rapid and reliable detection of SARS-CoV-2 antigens**
 - Results in 15 minutes
 - No cross-reactivity with minor coronaviruses (229E, OC43, NL63 & HKU1), Influenza A & B and RSV
- **Convenient**
 - The kit can be stored at room temperature or refrigerated (2-30°C)
 - No need for additional equipment : Everything is included in the kit
- **CE marked**

THE SOLUTION


COVID-19 is a disease caused by a novel coronavirus (CoV) SARS-CoV-2 also known as 2019-nCoV. In order to facilitate patient management and to identify infected individuals, especially those at risk of complications, it is essential to be able to rapidly detect the presence of SARS-CoV-2. The **BIOSYNEX COVID-19 Ag BSS** is a rapid chromatographic immunoassay for the qualitative detection of SARS-CoV-2 antigen, in only 15 minutes, using a nasopharyngeal or nasal swab.




PROCEDURE *




1 Open the **prefilled extraction tube**.

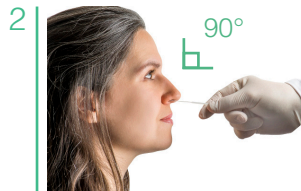

2 Carefully collect the sample in the **posterior nasopharynx** or in **both nasal cavities**.

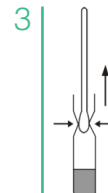

3 Discharge the swab into the extraction tube by **rotating** the swab for approximately **1 minute**.

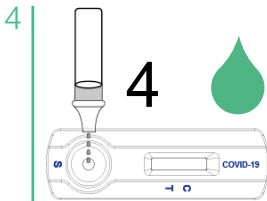


4 Screw on the cap of the extraction tube and place **4 drops** of the solution in well S.



5 Read the result at **15 minutes**. Do not read after 20 minutes.

* For more information please read the IFU

RESULTS

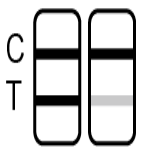

POSITIVE



NEGATIVE

KIT CONTENTS

BIOSYNEX COVID-19 Ag BSS

Ref. SW40006

- 25 test cassettes
- 25 sterile swabs
- 25 extraction tubes prefilled with extraction buffer

- 25 nozzles with filter
- 1 workstation
- Instructions for use

Controls available separately:

Positive control swab SW40009

Negative control swab 6060001



BSX RANGE

Also available as a reference compatible with the BSX READER® for automated cassette reading and full traceability (operator, patient, result).

- Ref SW40006_BSX - BIOSYNEX COVID-19 Ag BSX - 25 tests
- Ref 5060029 - BSX READER® - Quick test reader + power cable + barcode reader + EN/FR user manual
- Ref 502250 - DYMO printer LabelWriter 450 (more informations on request)

Manufacturer : BIOSYNEX Swiss S.A. (Switzerland)

In vitro diagnostic medical devices.
Read the instructions for use carefully.

FLY_SW40006_EN_V02202008R06
Rev : 06/21

BIOSYNEX

Distributor : BIOSYNEX S.A.

22 Boulevard Sébastien Brant

67400 ILLKIRCH-GRAFFENSTADEN - France

Phone: +33 3 88 77 57 52 - Fax: +33 3 88 78 76 78

E-mail : export@biosynex.com

www.biosynex.com